

Nasal solution - 20 ml bottle

INDICATIONS:

NARLISIM® Spray is a solution for nasal use, indicated in case of obstruction of the nostrils to facilitate the elimination of mucus; NARLISIM® Spray can also be used as a supplement in the treatment of the most common infections of the respiratory system.

COMPOSITION:

Pyrometaxine (pyrrolidonecarboxylic acid, copper sulfate pentahydrate), sodium chloride and purified water.

INSTRUCTIONS FOR USE:

For the correct use of the NARLISIM® Spray medical device, do the following:

- Remove the cover
- Tilt your head back slightly and insert the tip of the bottle into your nostril.
- Press the tabs to spray the dose into the nostril.
- Repeat the same for the other nostril.
- Lift your head and wipe off any excess liquid.
- Avoid blowing your nose immediately after administration of the product, in order to prolong the effect.
- Close the bottle and store in a dry place, away from heat sources and protected from light.

DOSAGE:

Children over 3 years old: 1 nebulization in each nostril, 2-3 times a day

Adults: 1 - 2 nebulizations in each nostril, 3-4 times a day

WARNINGS:

- NARLISIM® Spray is for nasal use only. Do not ingest.
- Use a spray for each person.
- Do not use in case of hypersensitivity or incompatibility with any component of NARLISIM® Spray. A mild and temporary burning sensation can be felt in the application and is associated with the characteristics of solutions needed to ensure effectiveness.
- Use only on intact mucosa.
- Do not use during pregnancy.
- In children under the age of three, NARLISIM® Spray should be used under medical supervision.
- Prolonged use of NARLISIM® Spray, as with all topical products, can give rise to local irritation and sensitization, if this happens, stop the treatment and consult your doctor or pharmacist.
- Do not use NARLISIM® Spray if, when opening the box for the first time, it appears to be damaged or not intact.

- Do not use NARLISIM® Spray after the expiration date shown on the box.
- Keep NARLISIM® Spray out of the sight and reach of children.

CONSERVATION:

Store in a dry place, away from heat sources (recommended temperature between 10 and 30 ° C) and protected from light. The expiration date indicated refers to the closed NARLISIM® Spray, properly stored. The expiration date is indicated on the packaging.

SECUNDARY EFFECTS:

There are no known or expected side effects related to the use of NARLISIM® Spray. The tests carried out to date have provided no evidence of sensitization or other side effects. If you feel pain, consult your doctor.

NARLISIM® Spray is a medical device classified and produced in compliance with the requirements of Council Directive 93/42 / EEC and subsequent amendments.

Last review date: November 2016